Panasonic[®]

Operating Instructions 使用說明書

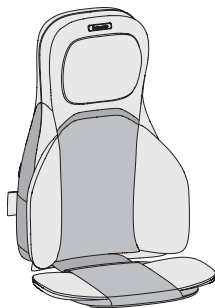
Massage Cushion
按摩背墊

Model No.
型號

EW-RADK4

English EN2

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Thank you for purchasing this Panasonic product.

Before operating this unit, please read these instructions completely and save them for future use.

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Contents

Important safety instructions	EN3
Part names and functions of main unit	EN8
Preparation	EN10
How to use.....	EN11
After massage is finished	EN17
Maintenance method	EN18
Troubleshooting	EN21
Specifications	EN22

Important safety instructions

Always follow these instructions

This section contains instructions that you must follow to prevent personal injury or damage to property.

- The instructions are classified according to the level of injury or damage that may occur due to the mistaken use of this unit. Please ensure that you read all instructions before using the Household Massage Lounger.

WARNING Denotes a potential hazard that could result in serious injury or death.

CAUTION Denotes a hazard that could result in minor injury or property damage.

- Instructions that you must follow are indicated using the following symbols.

Denotes a specific operating procedure that must not be performed.

Denotes a specific operating procedure that must be performed.

WARNING

Symptoms

- The following persons should not use this unit.

- (1) Persons medically prohibited from massage (e.g., those with thrombosis/embolism, severe aneurysms, acute venous aneurysm, various dermatitides, and skin infections [including subcutaneous tissue inflammation])
 - (2) Persons suffering from osteoporosis, spine fracture, sprain or acute pain such as a pulled muscle
 - (3) Persons who have a pacemaker, defibrillator, or other personal medical devices prone to electronic interference
 - (4) Pregnant women and women who have just given birth
 - (5) Persons who have abnormalities or curvature of the spine
 - (6) Persons with symptoms of acute low back pain, herniated disk, spondylolisthesis, spinal stenosis, degenerative lumbar spondylosis
- (Otherwise it may worsen accidents or symptoms.)

Important safety instructions (continued)

WARNING

Symptoms

- Persons (including children) with physical, sensory or mental disabilities, or lacking operational experience or knowledge, shall not use this unit without supervision and guidance regarding the use of this unit from a guardian. Children must be strictly supervised to prevent them from playing with this unit.

(Otherwise it may lead to accident or injury.)

- Persons with any of the conditions listed below or who are undergoing medical treatment should consult with their physicians before using this unit.
 - (1) Persons who have a malignant tumor
 - (2) Persons suffering from heart disease
 - (3) Persons who have a loss of heat sensation
 - (4) Persons who have sensory abnormalities due to a severe peripheral circulatory disturbance such as diabetes mellitus
 - (5) Persons who have a wound where the massager operates
 - (6) Persons whose body temperature is over 38 °C (who have a fever) (E.g.: When a person has severe acute inflammation symptoms [fatigued, chills, blood pressure fluctuations, etc.] or when debilitated)
 - (7) Persons who require bed rest or who are in poor physical shape
 - (8) Persons whose muscles have deteriorated due to age or weight loss
 - (9) Persons with symptoms of lower back and hip pain caused by the bones and muscles, organs and nerves
 - (10) Persons who suffer bruises or sprains easily
 - (11) Persons who suffer from severe motion sickness
 - (12) Persons who have undergone heart or internal surgery in the past
 - (13) Persons other than those listed above who feel unwell
- Discontinue use immediately and consult a physician if you experience physical discomfort or fail to perceive the intended therapeutic effects from this massager
- The unit contains heated surfaces. Users with reduced thermal sensitivity must exercise extreme caution during operation.

(Otherwise it may lead to accident, injury or become unwell.)

WARNING

When in use

- Do not use this unit while covered by a quilt or blanket. Do not use this unit inside a quilt.
(Otherwise it may lead to internal overheating of this unit body and lead to accidents.)
- Do not use this unit for any purpose other than massaging shoulders, neck, waist, or buttocks.
(Otherwise it may lead to accident or injury.)
- Do not use the "Heat" function for extended periods. Avoid continuous use after 15-minute massage.
(Otherwise it may cause injury or low-temperature scald.)

* Prolonged exposure of the same skin area to relatively low temperatures (40 °C to 60 °C) can cause burns, even if you don't feel heat or pain. Elderly persons, persons with delicate skin, or persons with reduced skin sensitivity may experience skin redness during initial use of this unit. If the redness persists after 1 hour, please consult a physician.

- Please control the time of each massage within 15 minutes.
- Use no more than twice a day, with approximately 30 minutes of massage per application.
(Otherwise, it may lead to opposite effects or injuries.)
- Before using this unit, be sure to check whether the fabric of this unit is damaged. (No matter how small the rip, if you find one then immediately stop using this unit, unplug the power adapter and have this unit repaired at the customer service center.) Continuing to use the fabric in a damaged condition may result in injury or electric shock accidents.

Power plug, power cord, etc.

- Do not perform any actions that may damage the power cord, power plug, and power cord plug. Do not scratch, modify, place near heaters, excessively bend, twist, pull, place heavy objects on, or bundle the power cord, power plug or power cord plug.
(Otherwise it may lead to electric shock, ignition or fire triggered by short circuits.)
- When unplugging the power cord, do not pull the cord.
(Otherwise it may lead to electric shock, ignition or fire triggered by short circuits.)
- Do not plug/unplug the power plug with wet hands.
(Otherwise it may lead to electric shock.)
- If the power cord is damaged, it must be replaced by a professional from the manufacturer, or the service department thereof, or a similar department in order to avoid hazards.

- Use sockets and wiring devices with correct rated values.
- Use 220 V (not available overseas, and transformer usage is prohibited.)
- Fully insert the power plug into the power outlet.
(Otherwise it may lead to electric shock or fire due to overheating.)

Important safety instructions (continued)

WARNING

If a malfunction or breakdown occurs

- If a malfunction or breakdown occurs, stop using the chair immediately and unplug the power plug from the wall socket. (Otherwise it may lead to smoke/fire or electric shock.)
<Examples of malfunctions/breakdowns>
 - Loose power cord plug
 - The device sometimes operates and sometimes fails to operate after the power switch is turned on
 - The power supply goes on and off when the power cord is moved
 - Burning smells or abnormal sounds occurring during operation
 - The unit becomes deformed or unusually hot
- Do not attempt to repair this unit by yourself. Please contact the customer service center for inspection and repair immediately.

Take note of the following points as well

- Do not puncture the fabric with pins or needles, or cut the fabric with knives. (Otherwise it may lead to fire, electric shock or scald due to damage to the warming position (heater) on this unit.)
- Do not detach the zipper from the fabric without authorization.
- Do not modify or disassemble this unit.
(Otherwise it may lead to fire, or cause this unit to function abnormally resulting in injury.)

CAUTION

Symptoms

- If rash, redness or itching occurs during use of this unit, discontinue use of this unit and consult a physician.
(Otherwise it may lead to accident or physical discomfort.)

Power supply

- Always unplug the power plug from the wall socket before cleaning or maintenance.
(Otherwise it may lead to electric shock or scald.)
- Please remove dust and other dirt from the power cord plug and the power cord socket periodically.
(Wipe away dust with a dry cloth. Otherwise it may lead to insulation degradation due to dampness and result in fire.)
- Always unplug the power plug from the wall socket when this unit is not used.
(Dust buildup combined with dampness may cause insulation degradation, leading to electrical faults and triggering a fire.)

CAUTION

Before and during use

- Do not stand or sit on this unit body.
- Do not apply excessive force or lean your body against the massager during use.
(Otherwise, it may lead to faults or accidents.)
- Do not unplug the power cord plug suddenly during massage.
(Otherwise it may lead to injury.)
- Do not wrap this unit body with cloth or other materials before use.
(Otherwise it may lead to injury.)
- Do not use this unit in high-humidity areas such as bathrooms.
(Otherwise it may lead to electric shock.)
- Do not spill water or other liquids on this unit.
(Otherwise it may lead to an electric shock accident or fire triggered by short circuits.)
- Do not use this unit while sleeping.
- Do not use this unit while intoxicated.
(Otherwise it may lead to accident or injury.)
- Do not place the massage head directly against your hands, elbow joints, knees, chest or abdominal areas.
- Do not allow pins, debris, water, etc. to enter the gaps of the power cord plug and power cord socket.
- Always unplug the power plug from the wall socket when this unit is not used.
(Otherwise, it may cause electric shock, short circuit, burn or fire.)
- During storage, do not coil the power cord around the power plug or unit body.
(Otherwise, it may overload the power cord and cause power cord breakage, resulting in electric shock or short circuits.)
- Do not use at the same time as another medical device.
(Otherwise it may lead to accidents or affect physical health.)

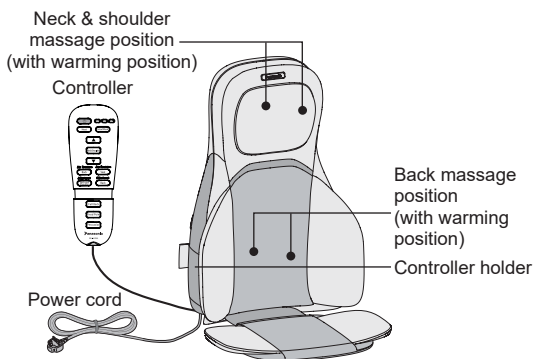
- If this unit has not been used for a period of time, read the instruction manual carefully again to confirm normal functioning of this unit before starting use.
(Otherwise it may lead to accident or injury.)
- Prior to use, please confirm first that no foreign objects are present between any components of this unit.
(Otherwise, it may trap shoulders, neck, waist, etc., causing accidents or injuries.)
- In the event of a power outage, unplug the power plug immediately.
(Otherwise it may lead to accident.)
- Remove necklaces and other accessories before massage.
(Otherwise it may lead to injury.)


Part names and functions of main unit

Main body


<Front>

<Back>

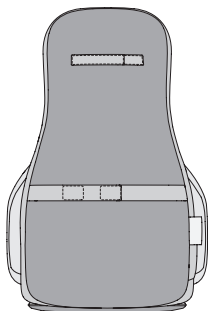Operation position

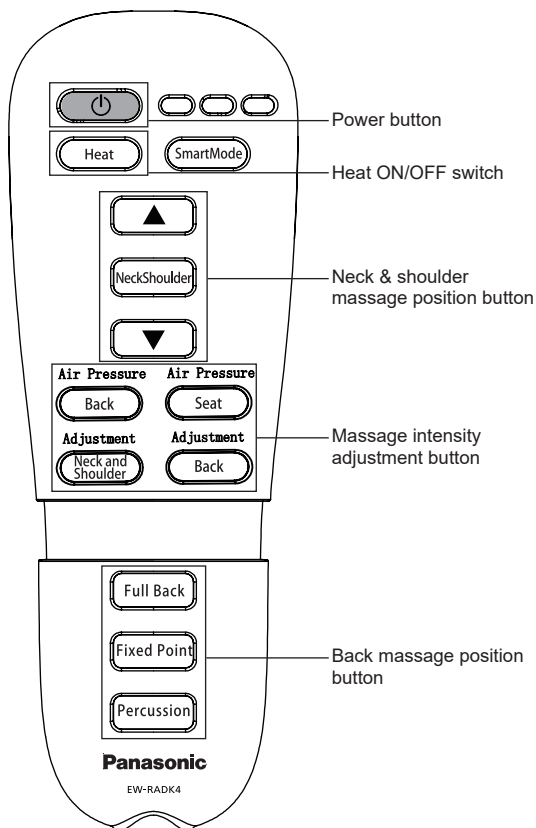
Part names and functions of main unit


Preparation

1 Confirm if the fabric is damaged

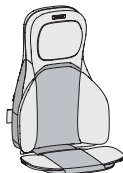- No matter how small the rip, if you find one then immediately stop using this unit, and have this unit repaired at the customer service center.


2 Place the product on a chair

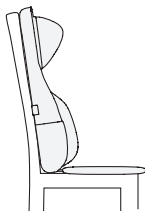How to use

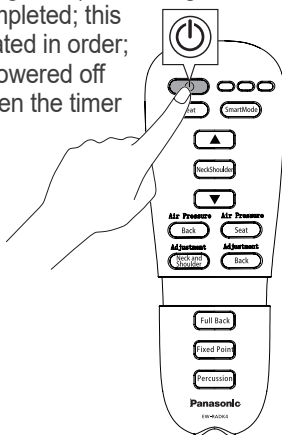- 1 When the  ON/OFF switch is pressed, this unit is powered on, the red indicator lights up, all other button lights remain off, and a default 15-minute timer is activated; when the ON/OFF switch is pressed again, this unit is powered off, all functional motors are reset in the massage process when the power is turned off (the red light flashes during reset), and all lights are off once reset is completed; this process is circulated in order; this unit will be powered off automatically when the timer expires.

Preparation

How to use

2 Functions of different buttons

● Heat button

This feature is active when the neck and shoulder or back massage function is activated when this unit is powered on:

When the Heat button is pressed once, heating is activated, and the red light of the Heat button is illuminated; when the Heat button is pressed again, heating is deactivated, and the red light is off.

Back heating remains inactive unless the back movement structure is activated. Neck and shoulder heating remains inactive unless the neck and shoulder movement structure is activated. This process is circulated in order;

EN11

How to Use (continued)

● Smart Mode button

This feature is active when this unit is powered on:

The first press activates Mode 1, and the indicator light corresponding to Mode 1 is illuminated.

The second press activates Mode 2, and the indicator light corresponding to Mode 2 is illuminated.

The third press activates Mode 3, and the indicator light corresponding to Mode 3 is illuminated.

This process is circulated in order;

Mode 1:

1. Activate neck and shoulder kneading/heating first at default low intensity; simultaneously activate full back/ heating/percussion massage on your back at default low intensity; simultaneously activate seat and back airbags at default low intensity
2. Activate neck and shoulder kneading/heating at medium intensity; simultaneously activate percussion/kneading/heating on the lower back area, switch to fixed-point percussion at medium intensity after 1 minute while deactivating kneading and heating; simultaneously activate seat and back airbags at default medium intensity.
3. Activate clockwise and counterclockwise neck and shoulder kneading/ heating at high intensity; simultaneously activate upper-back percussion/kneading/heating at high intensity; simultaneously activate seat and back airbags at default high intensity.

This process is circulated in order

Smart Mode 2:

The neck and shoulder massage proceeds to the lowest part of the neck and shoulder position, with the indicator light illuminated;

1. Activate neck and shoulder massage only at default low intensity of both heating and massage; simultaneously activate the seat airbag at low intensity.
2. Then activate back kneading/heating combined with moving massage nodes that traverse along the spine (without percussion) at default low intensity; simultaneously activate the back airbag at low intensity.
3. Change neck and shoulder massage intensity to medium; activate the seat airbag at medium intensity.
4. Then activate back percussion/kneading under default heating at high intensity; simultaneously activate seat/back airbags at high intensity.

This process is circulated in order.

Smart Mode 3:

The neck and shoulder massage proceeds to the uppermost part of the neck and shoulder position, with the indicator light illuminated;

1. First activate back kneading/heating/percussion/back airbag at default low intensity;
2. Then activate the neck and shoulder kneading/heating/seat airbag at default low intensity;
3. Fully activate back + neck and shoulder functions at medium intensity;
4. Fully activate back+neck and shoulder functions at high intensity.

This process is circulated in order

● Neck and Shoulder button

This feature is active when this unit is powered on:

The Neck and Shoulder button light is simultaneously illuminated

First press of this button activates clockwise neck and shoulder kneading massage with the indicator light illuminated;

Second press of this button activates counterclockwise neck and shoulder kneading massage with the indicator light illuminated;

Third press of this button activates automatic clockwise and counterclockwise neck and shoulder kneading massage with the indicator light illuminated;

Fourth press of this button deactivates the neck and shoulder massage function, and the indicator light goes out;

This process is circulated in order

● Neck and Shoulder Fixed Point Up button

This feature is active when this unit is powered on:

Long press on this button to make the neck and shoulder movement structure move upward, and release this button to stop the movement structure. If the upper limit is not reached, the blue light flashes slowly; if the upper limit is reached, the movement structure stops, and the blue light remains steadily illuminated.

● Neck and Shoulder Fixed Point Down button

This feature is active when this unit is powered on:

Long press on this button to make the neck and shoulder movement structure move downward. If the lower limit is not reached, the blue light flashes slowly; if the lower limit is reached, the movement structure stops, and the blue light remains steadily illuminated; if this button is released, the movement structure stops, and the blue light goes out.

● Neck and Shoulder Adjustment button

This feature is active when the neck and shoulder kneading mode is activated when this unit is powered on:

First press of this button activates neck and shoulder kneading at medium speed, with the corresponding indicator light flashing rapidly;

Second press of this button activates neck and shoulder kneading at high speed, and the corresponding indicator light remains steadily illuminated;

Third press of this button activates neck and shoulder kneading at low speed, with the corresponding indicator light flashing slowly;

This process is circulated in order;

How to Use (continued)

● Back Air Pressure button

This feature is active when this unit is powered on:

If Seat Air Pressure is not activated:

First press of this button activates the back inflation mode at default low intensity, with the corresponding indicator light flashing slowly;

Second press of this button activates the back inflation mode at medium intensity, with the corresponding indicator light flashing rapidly;

Third press of this button activates the back inflation mode at high intensity, and the corresponding indicator light remains steadily illuminated;

Fourth press of this button exits the back inflation mode, and the corresponding indicator light goes out;

This process is circulated in order;

If the seat inflation mode is activated:

Press the Back Air Pressure button to activate the back and seat inflation mode while maintaining the current intensity;

Press the Back Air Pressure button again to switch the inflation intensity levels. If the previous intensity is high, this will deactivate the back inflation mode while maintaining the current intensity.

● Seat Air Pressure button

This feature is active when this unit is powered on:

If Back Air Pressure is not activated:

First press of this button activates the seat inflation mode at default low intensity, with the corresponding indicator light flashing slowly;

Second press of this button activates the seat inflation mode at medium intensity, with the corresponding indicator light flashing rapidly;

Third press of this button activates the seat inflation mode at high intensity, and the corresponding indicator light remains steadily illuminated;

Fourth press of this button exits the seat inflation mode, and the corresponding indicator light goes out;

This process is circulated in order;

If the back inflation mode is activated:

Press the Seat Air Pressure button to activate the back and seat inflation mode while maintaining the current intensity;

Press the Seat Air Pressure button again to switch the inflation intensity levels. If the previous intensity is high, this will deactivate the back inflation mode while maintaining the current intensity.

● Back Adjustment button

This button is active only when the back percussion mode/kneading mode is activated.

First press activates back percussion/kneading at medium speed with the indicator light flashing rapidly;

Second press activates back percussion/kneading at high speed, and the indicator light remains steadily illuminated;

Third press activates back percussion/kneading at low speed with the indicator light flashing slowly;

This process is circulated in order;

● Full Back button

This feature is active when this unit is powered on:

First press of this button activates the full back massage function, with the movement structure performing vertical traversal motion and kneading at medium intensity, and the corresponding indicator light is illuminated;

Second press of this button deactivates the full back massage function, and the corresponding indicator light goes out;

This process is circulated in order;

● Fixed Point button

This feature is active when this unit is powered on:

First press of this button activates the back Fixed Point function, with kneading at medium intensity and the movement structure performing vertical traversal motion, and this process is circulated in order; the corresponding indicator light is illuminated;

Second press of this button deactivates the back Fixed Point function, and the corresponding indicator light goes out;

This process is circulated in order;

● Percussion button

This feature is active when this unit is powered on:

First press of this button activates the back percussion function, and the corresponding indicator light is illuminated;

Second press of this button deactivates the back percussion function, and the corresponding indicator light goes out;

This process is circulated in order;

How to use

How to Use (continued)

Example massage positions

Neck and shoulder/back/buttocks

Place the product on a chair or sofa for use

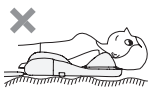
● Do not use the product in the following ways


Do not wear this unit and lie prone on this unit body during use

Do not wear this unit and lie on this unit body during use

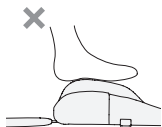Do not use while lying on uneven surfaces during use

Do not use for foot massage

To enhance massage sensation

- Use the product on stationary-back chairs or sofas as far as possible.
- Sit as close to the product as possible, slightly straighten your upper body, and maintain full contact with this unit body to achieve the effect of enhanced massage sensation.

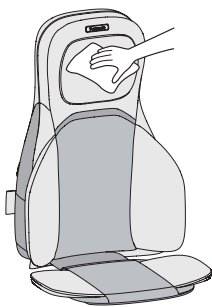
Maintenance method

Be sure to perform the following operations after power-off

- 1 Dampen a cloth with water and squeeze out excess water
(When the massage cushion is heavily soiled, dilute kitchen neutral detergent to approximately 5 % solution with warm water or water for use.)

- 2 Wipe away dirt
 - Avoid applying excessive force.


- Never use chemicals such as thinner, benzene, alcohol, etc. to clean this unit.

3 Neck and shoulder fabric cover: Water washable after removal.


Removal

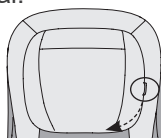- ① Disengage the zipper slider from the teeth.
- ② Pull the zipper in the direction shown.
- ③ After pulling the zipper open, remove the neck and shoulder fabric cover.

Installation

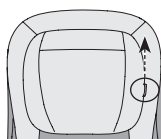- ① Secure the slider in locked position.
- ② Pull the zipper in the direction shown.
- ③ Retract the slider into designated storage cavity.

Laundry Instructions

(Laundry symbols)

Maintenance method

■ During hand washing (Do not use a washing machine)

- Wash with laundry mesh bag.

① Immerse in water below 30 °C and wash by gentle pressing

Note

- No scrubbing.
- Bleach-containing detergents are forbidden to prevent color fading.
- Please wash separately from other items due to dye transfer risk.

② Flush thoroughly

③ Gently press to drain moisture

Note

- Wringing prior to spinning is strictly prohibited to prevent fabric damage or yarn abrasion.

Maintenance method (continued)

④ Gently smooth out wrinkles and dry in shade

Note

- Sunlight exposure is strictly prohibited to prevent fabric shrinkage or fading.
- Dry in shade immediately after washing as the color may stain other items easily.

Steam ironing to remove wrinkles is permitted after washing.
Iron usage instructions

Please press with a protective cloth, and iron at medium temperature of not exceeding 160 °C.

■ During dry cleaning

For entrusted dry cleaning (petroleum dry cleaning system) at the dry cleaner's, please remind the cleaner to observe the following precautions to prevent fabric damage.

- Be sure to use laundry mesh bag.
- Gentle washing only. Do not tumble dry.

Troubleshooting

- Please perform the following inspections. If the problem still cannot be solved, discontinue use immediately to prevent accidents and entrust a service center authorized by Panasonic for repair.
- Do not attempt to disassemble or repair the product by yourself.

Abnormal condition	Possible cause	Solution
The complete device exclusively performs upper back or lower back massage	The controller has been configured with semi-back control	Press the Full Body/Upper Back/ Lower Back button to change massage areas
Inoperable product	<ol style="list-style-type: none">1. Power supply not properly connected2. Standby button not activated3. The movement structure securing screw at the back of this unit is not removed	<ol style="list-style-type: none">1. Test the power plug, connect power supply and press the Standby button2. Remove the screw using the hex wrench provided in accessories
Massage headlight not illuminated (Heat function inoperative)	Heating duration exceeds 15 minutes or Heat button is not activated	Reactivate the Heat button

Maintenance method

Troubleshooting

Specifications

Product name	Massage Cushion
Product model	EW-RADK4
Rated voltage	220 V ~ 50 Hz
Rated power	70 W
Time	Approx. 15 minutes
Dimension (D × W × H)	Approx. 805 mm × 225 mm × 480 mm
Mass	Approx. 13.1 kg
Manufacturer	Panasonic Manufacturing (Shanghai) Co., Ltd. No. 258, Jiangtian East Road, Songjiang Industrial Park, Shanghai, China
Country of origin	China

This image shows a blank sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

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© Panasonic Corporation 2025
Panasonic Corporation